

# ELLIOCATH N.V.®

## Embryo transfer catheter (metal free)

### PRODUCT DESCRIPTION

- Curved transfer catheter making easier the passage through the cervical canal.
- Tactile marks on the grip of ELLIOCATH N.V.® indicate the direction of the curve of the catheter.
- Set of 4 labels for traceability.
- Centimetric graduation on 360°.
- Tight sliding ring.

### INDICATIONS

- Embryo transfer
- Intra Uterine Insemination

### RECOMMENDATIONS FOR USE


- Before use, wash carefully the catheter with an appropriate sterile culture medium.

#### Embryo transfer

- Place the embryo in ELLIOCATH N.V.® with a small volume of culture medium.
- Position the sliding ring according to the uterus depth previously measured.
- Advance ELLIOCATH N.V.® to a distance 1 cm from the uterine fundus.
- To facilitate cervical canal passage, push the catheter with a gentle rotational movement.
- Transfer embryo.
- Slowly withdraw the catheter

#### Intra uterine insemination

- Push ELLIOCATH N.V.® into the uterus.
- To facilitate cervical canal passage, push the catheter with a gentle rotational movement.
- Push slowly the sperm preparation.
- Slowly withdraw the catheter.

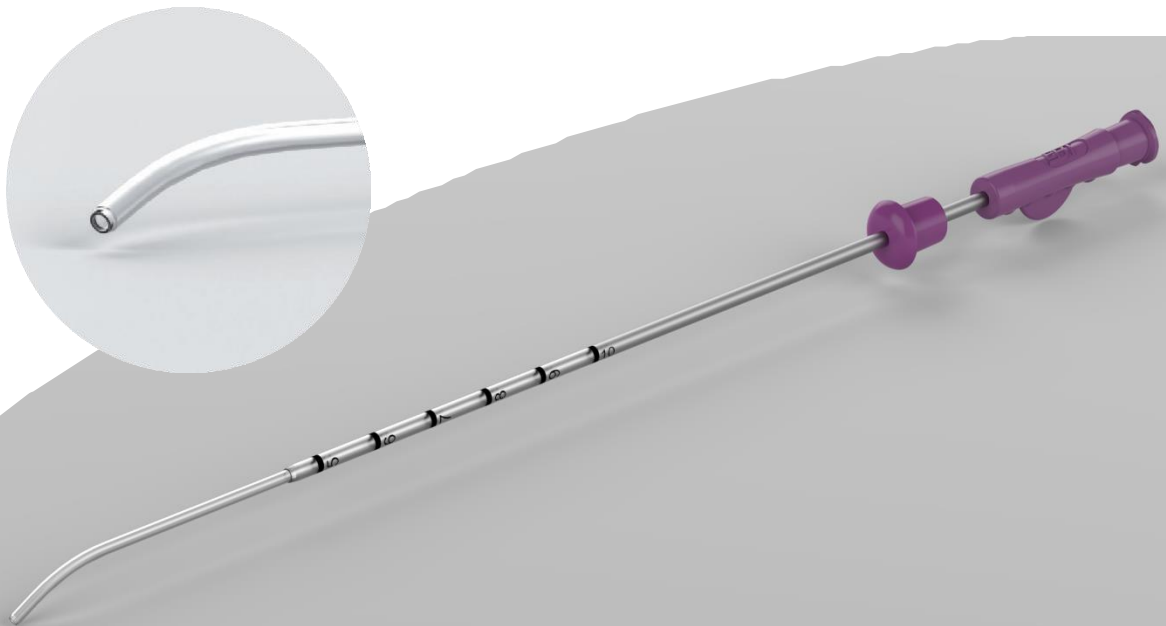

Référence	Désignation	Conditionnement	Ø int.	Longueur	Stérilisation
EBT018-KT-NV	ELLIOCATH N.V.®	Boîte de 25 unités Sachet individuel	1.1 mm	20.5 cm	γ irradiation